

## Fitting Instructions



*I've gotten so many comments  
on my beautiful smile!*

Once fitted, snap them on anytime without denture adhesives!  
No dental appointment necessary.

Not a therapeutic device. Not for braces.  
Patent #5,951,291 ©2012: Dr. Bukk, Inc.

Being embarrassed to smile affects your quality of life  
profoundly. It can limit your career and social life.



Imako is not a therapeutic device but it can lift your  
self esteem for an affordable price!


Thank you for purchasing Imako Cosmetic Teeth!



*The teeth arrive flat.*



*Once fitted they look like this.*

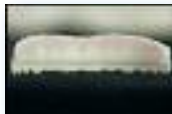

You will change this device from flat to arch-shaped and springy. It fits mechanically, not by being sticky.

*A Tiny ledge under front bite surfaces stabilizes position.*

*B Points between teeth lock device on to your teeth.*

*C Imako flexes outward to put on and take off.*

- You need a mirror, a timer with seconds displayed, a pot of simmering water, a ceramic bowl and some scissors/nail clippers.
- Wash hands, floss & brush teeth before beginning.

*The back layer, the white beads and the pink chunk are the same material, which will go back and forth: soft when hot or solid under 120°. It will bond to itself when a hot surface touches a hot surface.*

*First, you will create a pink ramp at the top edge so your lip will not catch on the device. This strengthens the fit substantially.*

*Avoid letting soft fitting material touch painted fingernail surfaces.*

After you have completed the fitting process, the Imako Cosmetic teeth® will be arch shaped and springy. You can snap them into position any time. It does not feel tight but stays right in place.

The Imako teeth can be discreetly removed by jutting your lower jaw forward and catching the lower edge while pretending to cough into a napkin.

Nothing on the market is thinner!



- Soften the chunk of pink plastic in a bowl full of very hot water. Remove it with a spoon.

- Roll it out on your palm until you have a thin tube the size of a spaghetti noodle the width of your palm (you have more than necessary).

- If needed keep soft by dipping in hot water.



- Turn the Imako® teeth upside down and dip briefly in the hot water to make it receptive to bonding.



- Attach the pink spaghetti piece to the top edge of the back layer.

- Place the teeth face down in the hot water. You may need to refresh the water. When the back layer turns completely clear, pour off the water.

- Now the teeth should be very floppy. Pull your upper lip forward and insert into your mouth. Look in a mirror and get them centered and level.



· Press one time, very firmly against each of your teeth. Gently make sure the pink material is making contact with your gums. The device will not work immediately. If you continue to press hard you will slide it and ruin the imprint. Make sure your back teeth are properly hidden.



**Softly support the position for TWO MINUTES and no longer!**

· Remove the teeth by spreading from below and place it on the countertop. Allow it to cool a couple of minutes. Once the back layer turns white, snip off the tips of the points which are formed where two teeth meet the gum, or they will irritate your gums.

*Do not allow the device to completely set in your mouth!*

*If device will not remove easily, swish your mouth with the hottest water you can withstand to melt fitting material, scrape away bit by bit. DO NOT use force.*

## The White Beads - What Are They For?

If you have missing teeth, you have the option to use the white beads to create projections on the inside surface of the Imako® Teeth.

Making the projections will do two things. First it will act as a spacer to keep your other teeth from moving. Second, it acts like the male portion of a snap fastener and gives a more secure fit.



Soften two beads per missing tooth in hot water. Dip the Imako® area where you have no tooth print briefly in hot water so the beads will bond in that spot. Position in your mouth and press the warm plastic to butt up to neighboring teeth, **but not behind them**. Allow to cool until material is just slightly soft and remove.

You can easily add more but it is hard to take away so be conservative.

If you have many teeth missing on one side, you can wrap some fitting material halfway around the last tooth. You will need to put this side on first and take it off last. Where your teeth are missing add more beads to stiffen the area.

If you have no side teeth at all, bond two or three beads along the bottom edge where you have teeth and fold the material around the back side. Press it out to be **very thin** so it is flexible. This may obstruct closing your teeth.



*The Imako Cosmetic Teeth® are the thinnest, most comfortable temporary teeth.*

## Cleaning & Maintenance

Brush with your favorite toothpaste. Wipe with rubbing alcohol each night if you depend heavily on your Imako® teeth. Soak in peroxide overnight from time to time.

Smoking will quickly cause stains. It is best to use our coating technique and apply the Imako® Dental Sealer after that, on both your own teeth and the cosmetic teeth surfaces. If tar stains appear, use denatured alcohol (found at hardware stores) to wipe away tar and brush.

## Coating Technique

We highly recommend coating with clear nail polish if you have a staining problem unless you are allergic to fingernail polish. After fitting, wash & let dry completely. Paint all surfaces of the front layer with clear nail polish. When dry and all aroma is gone, you must hit it with a flame to make the polish sink into the teeth or it will crack and peel off.



Use a lighter with a high flame. Hold the teeth below. Rotate your wrist so your thumb is positioned below the flame when you point the flame downward. Let the flame touch and spread across all the surfaces briefly. This will permanently bond the polish.

### **Guarantee and Swaps**

Return to seller if their policy allows. If purchased direct, the Imako Cosmetic Teeth® can be returned within 30 days. If unused you get a full refund minus shipping charges. The product arrives inside a sealed, clear plastic sleeve, which if opened means the item is "used".

If you return used product, you receive \$24.95 as a refund. We keep \$15 as a restocking fee, because we must dispose of contaminated product to protect the public health.

Swapping is free if teeth are unused (still sealed in the clear sleeve). In a regular envelope, mail only the teeth wrapped in your receipt and write on there what you desire. A used teeth swap costs you \$15. Flatten used teeth in hot water and mail them in the same manner.



Watch our Imako Fitting Instructions Video on our web site at <http://www.imako.net/content/18-fitting-instructional-video>

or

Call for voice instructions press option 1



Thanks for your order!

Dr. Bukk, Inc.  
704 S. Old Belair Rd.  
Grovetown, GA 30813  
866-513-7308 or 706-855-1514  
<http://www.imako.net>  
Hours: 10-5:30 M-F EST